Mini Turbo Charger

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The Starkey Hearing Technologies Mini Turbo Charger has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated and conducted emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- Harmonic distortion and voltage fluctuations affecting the power input source as stated in Table 2 of IEC 60601-1-2.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.
- Immunity to electrical fast transients on the power input at a level of +/- 2 kV at a 100 Hz repetition rate.
- Immunity to surges on the power input of +/- 1 kV line to line.
- Immunity to conducted disturbances induced by RF fields on the power input as stated in Table 6 of IEC 60601-1-2.
- Immunity to voltage dips and interruptions on the power input as stated in Table 6 of IEC 60601-1-2.

FCC NOTICE

This device complies with part 15 of the FCC rules and with ISCED Canada license-exempt RSS standards. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this hearing aid must accept any interference received, including interference that may cause undesired operation.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

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Class II Device

Waste from electronic equipment must be handled according to local regulations

Consult Operations Manual

Keep dry

INSTRUCTIONS FOR DISPOSAL OF OLD ELECTRONICS

Starkey Hearing Technologies encourages, the EU requires, and your local community laws may require, that your hearing aids and charger be disposed of via your local electronics recycling/disposal process.

The instructions below are provided for the benefit of disposal/recycling personnel. Please include this manual when disposing of your hearing aids and/or charger.

FOR DISPOSAL/RECYCLING PERSONNEL ONLY

This charger contains a Lithium-Ion Polymer battery. To remove the battery from the charger, follow one of two options:

Option 1

- Insert pry bars between the plastic and the USB metal housing (top and bottom)
- Pry apart to break the seam
- Clip battery wires and dispose battery

Option 2

- Using the style groove as a guide, sever the plastic case on all 4 sides with a dremel blade, ultrasonic knife, hot wire, or band saw blade
- In each case, establish a limit guide so entrance over 3/32 inch beyond the outer shell is prevented
- Clip battery wires and dispose battery

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