Remote

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Wireless Technical Description

The Remote operates in the 2.4 - 2.4835 GHz band with a maximum effected radiated power of 3dBm with transmission modulation type of 1M06F1W. The receiver section of the radio has a bandwidth of 2 MHz.

The Remote has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated and conducted emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.1-1-2.

The Remote has a rating of IP 5X per IEC 60529. This means that the Remote is protected from dust.

The Remote does not contain any user-serviceable parts. Do not open or modify this device.

Do not connect the Remote to any external device.

The expected service life of the Remote is 3 years.

Wireless Information

KCC ID: MSIP-CRM-EOA-24WR

FCC ID: EOA-24WR

IC: 6903A-24WR (Model 610)

FCC/IC Notice

This device complies with part 15 of the FCC rules and with Industry Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

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Hereby, Starkey Hearing Technologies declares that Starkey Hearing Technologies Remote Microphone+ is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU. A copy of the Declaration of Conformity can be obtained from the following addresses and from docs.starkeyhearingtechnologies.com

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